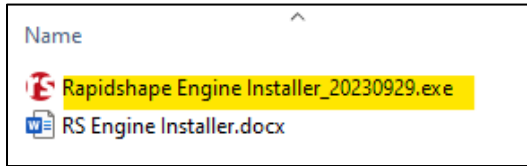
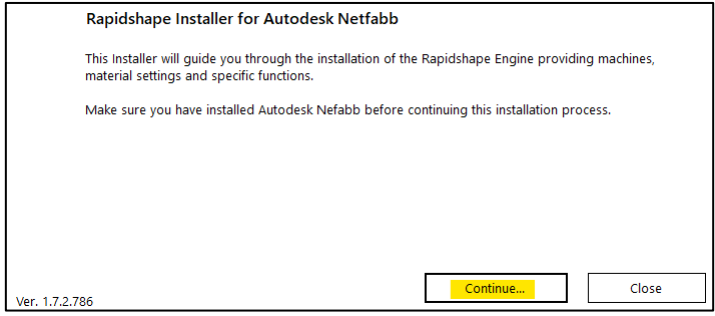
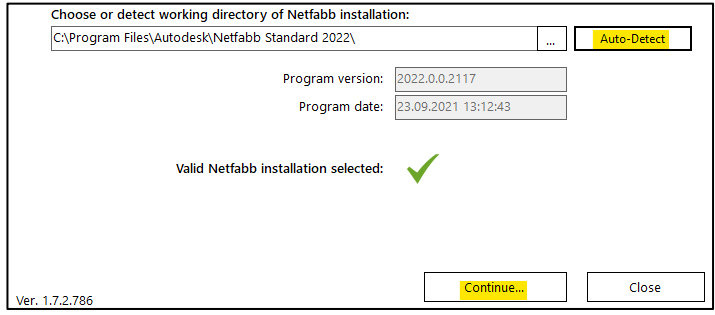


# Updating Netfabb material library RS Engine Installer

<p>0. To get the latest material library for your SHERAprint 3D printing system, you need the latest Rapidshape Engine (database)</p>	
<p>1. In the zip folder you downloaded right now, you will find an .exe file named Rapidshape Engine Installer</p>	
<p>2. To start the application please close all existing instances of Netfabb and double click the .exe file</p>	<p><i>Windows may warn you that you are about to run a potentially dangerous application. You can confirm this warning and start the application.</i></p>
<p>3. In the start up window press "Continue"</p>	
<p>4. Click „Auto-Detect“ – the Engine Installer should find your recently used Netfabb Version (most likely Netfabb 2022)</p> <p>Confirm with „Continue“</p>	

5. Now the recently used version („You have“) and the latest available version („You get“) of Netfabb is displayed.  
Confirm with „continue“

A Netfabb installation was detected successfully. Please check the information below to proceed:

You have		You get
Engine date:	18.07.2023 11:04:12	20.09.2023 17:26:10
Engine version:	2021.0.2054.778	2021.0.2054.786

[Release Notes](#)

Select your country: Germany

Autodesk login already available and license pre-registered.  
Use Autodesk login (not required)? No

Ver. 1.7.2.786

Back... Continue... Close

6. For „additional features“ check the box for „NetfabbPrint“ and click „Install“ for updating the database.

Select additional features:

☒ NetfabbPrint - Shell execution for Netfabb

☐ 3Shape Produce Integration

Call Rapidshape Engine for Netfabb via NetfabbPrint.exe from the command line to load STL files or archives (e.g. RShapeSTL) and execute a suitable workflow.

Use this feature to establish interfaces from other CAD/CAM applications (e.g. exoprint).

Ver. 1.7.2.786

Back... Install... Close

7. When the message „Installation successful!“ is displayed, you can click „Start Netfabb“.

24.11.2023 12:48:13 - ..... 'rapidshape.netfabbengine' copied to 'C:\Users\auer.SHERA\AppData\Roaming\Netfabb\rapidshape eng'  
 24.11.2023 12:48:13 - ..... 'workspaces.xml' copied to 'C:\Users\auer.SHERA\AppData\Roaming\Netfabb\rapidshape eng'  
 24.11.2023 12:48:13 - ... Backup finished  
 24.11.2023 12:48:13 - ... Installing Engine package...  
 24.11.2023 12:48:13 - ..... Copy 'rapidshape.netfabbengine' and 'workspaces.xml'...  
 24.11.2023 12:48:13 - ..... Installing feature 'NetfabbPrint - Shell execution for Netfabb'...  
 24.11.2023 12:48:13 - ..... Copied to 'C:\ProgramData\Rapidshape\NetfabbPrint'  
 24.11.2023 12:48:13 - ..... Added file 'printrequest\_to\_netfabbprint.bat'  
 24.11.2023 12:48:14 - ..... Validating installation for S-1-5-21-1420774829-969474738-973961551-4617...  
 24.11.2023 12:48:14 - ..... Registering engine for Germany...  
 24.11.2023 12:48:14 - ..... Setting Autodesk licensing option "Not using Autodesk login"...  
 24.11.2023 12:48:14 - ... Engine installed  
 24.11.2023 12:48:14 - ... Installation files deleted  
 24.11.2023 12:48:14 - **Installation successful!**

Ver. 1.7.2.786

Back... Start Netfabb... Close

8. New materials can now be added to your material library!